


**1** Insert the battery.




**2** Insert the code chip and compare the number to the number printed on the package, confirm the two are the same.

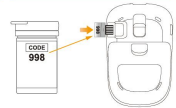

**3** Wash and dry your hands.




**4** Remove the lancing device cap.

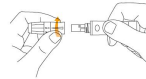


*Continued...*


**5** Insert a lancet into the lancing device.

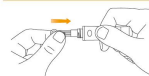


**6** Twist off the safety cap and put aside. Do not discard, it will be used later.

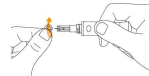


**7** Replace the lancing device cap.

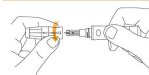

**8** Adjust the puncture depth: 1 is shallow, 5 is deep.

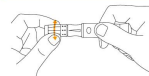

**9** Pull back the cocking barrel until you hear a click.

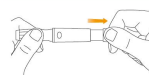

**10** Insert a test strip and confirm the code chip number.



**11** Holding the lancing device against the side of the finger, press the release button.



**12** Massage the finger to increase the sample size. Wipe away the first drop, and use the second drop to test.



**13** Apply blood to the sample area of the test strip. When blood fills the test window, the meter will count down.



**14** After 5 seconds, the result will display.



After the test, eject the strip using the test strip ejector, replace the lancet safety cap, and discard the lancet.