

Note: Depending on which GoChek2 control cap purchased, some of the components may need to be purchased separately. Please check the list of components printed at the back for details on what components are included with your purchase.

EC REP
Lettis Global Co., Ltd.
1 Fuyi Avenue, Tianjin
Tianjin, China 300160
Document: 10174BL-00010
Revision: V01
Issued: 20190422

MicroTech Medical (Ningbo) Co., Ltd.
Address: No. 9 Heiluo Road, Ningbo District
Ningbo, China 311121
Phone: +86-571-85565372
Website: <http://www.microtechmd.com>

1 ▶ Insert the battery.



2 ▶ Wash and dry your hands.




Insert a lancet into the lancing device.

3 ▶ Remove the lancing device cap.




4 ▶ Insert a lancet into the lancing device.

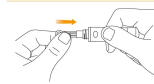


Continued...

5 ▶ Twist off the safety cap and put aside. Do not discard, it will be used later.

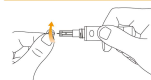


6 ▶ Replace the lancing device cap.

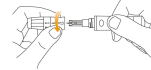

7 ▶ Adjust the puncture depth: 1 is shallow, 5 is deep.



8 ▶ Pull back the cocking barrel until you hear a click.

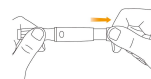

9 ▶ Insert a test strip.



10 ▶ Holding the lancing device against the side of the finger, press the release button.



11 ▶ Massage the finger to increase the sample size. Wipe away the first drop, and use the second drop to test.



12 ▶ Apply blood to the sample area of the test strip. When blood fills the test window, the meter will count down.



13 ▶ After 5 seconds, the result will display.



After the test, eject the strip using the test strip ejector, replace the lancet safety cap, and discard the lancet.